



# HEMOCOLL™


THE ABSORBABLE STERILE FIBRILLAR COLLAGEN WOUND FILLER  
FOR DENTAL SURGERY

Hemocoll is NON TOXIC AND NON PYROGENIC



**Hemocoll is high purity Type I reconstituted Collagen**

Hemocoll to be used in

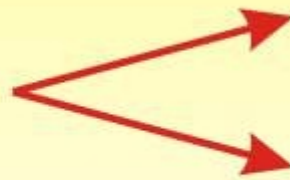

**EXTRACTION SITES**

**BIOPSY SITES**

Benefits of Hemocoll:

1. Controls bleeding and stabilizes blood clots (with in 2 to 5 Minutes)
2. Protects wound bed.
3. Helps in regeneration of lost Tissue.
4. Resorbable in 3-4 weeks.
5. Faster healing process.

**Hemocoll :** Available in the form of dry fibrillar collagen  
Individual pack size of 2.5 ml & 5 ml Steriled

**Precautions:**

- ◆ Hemocoll is not to be used on infected or contaminated wounds.
- ◆ Not intended for use in treating systemic coagulation disorders.
- ◆ Do not store above 32° C (90° F),
- ◆ Discard all open and un used dressing.
- ◆ Do not attempt to resterilize.
- ◆ Safety in pregnancy not established.

**Instruction for use :** Surface to be treated should first be compressed with dry gauze, remove the gauze carefully, and immediately cover the surface with Hemocoll. The amount required will depend on the force applied and severity of the bleeding. It is necessary to apply moderate pressure over the Hemocoll with a dry gauze.

Manufactured & Marketed by :

 **EnColl**

Fremont, CA, USA

Tel. : +1-510-659-1466

Fax : + 1-510-795-7571

E-mail : info@encoll.com

A Product with American Patented Technology  
(U.S. Pat. # 6548077, 6127143, 5814328)